


# Neuro Light Technical Set Up

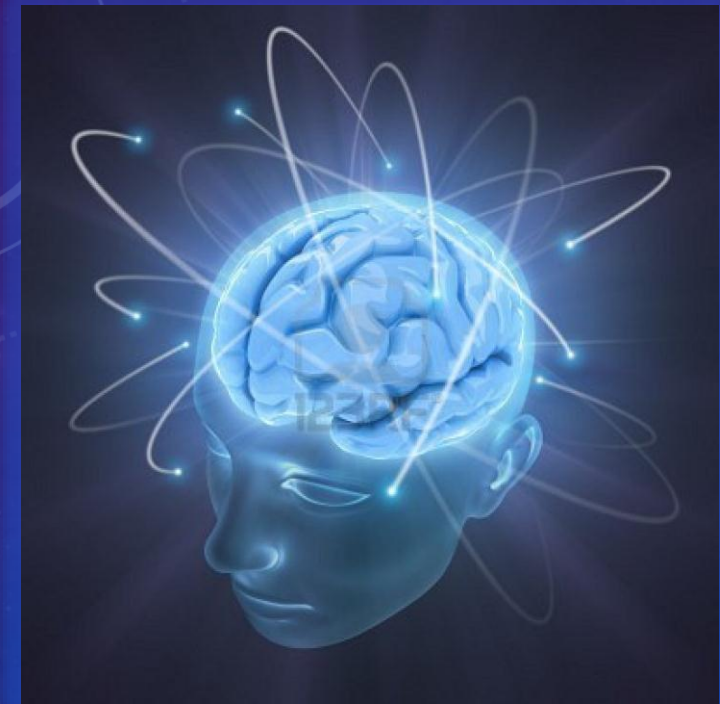

NEURO LIGHT

TOTAL BRAIN WELLNESS

# NEURO LIGHT TECHNICAL SET UP

- This **presentation** will take you **step by step** through the **technical Set Up** of your **Neuro Light device**.
- It is quite **simple and straightforward**.






# NEURO LIGHT TECHNICAL SET UP

- The **Neuro Light** has **two** related pieces of **hardware**.
- This shows the **Golden Pyramid** which houses the **5 super-luminous LEDs**.



# NEURO LIGHT TECHNICAL SET UP


- The **underside** of the **Golden Pyramid** has **5 super luminous LEDs**.

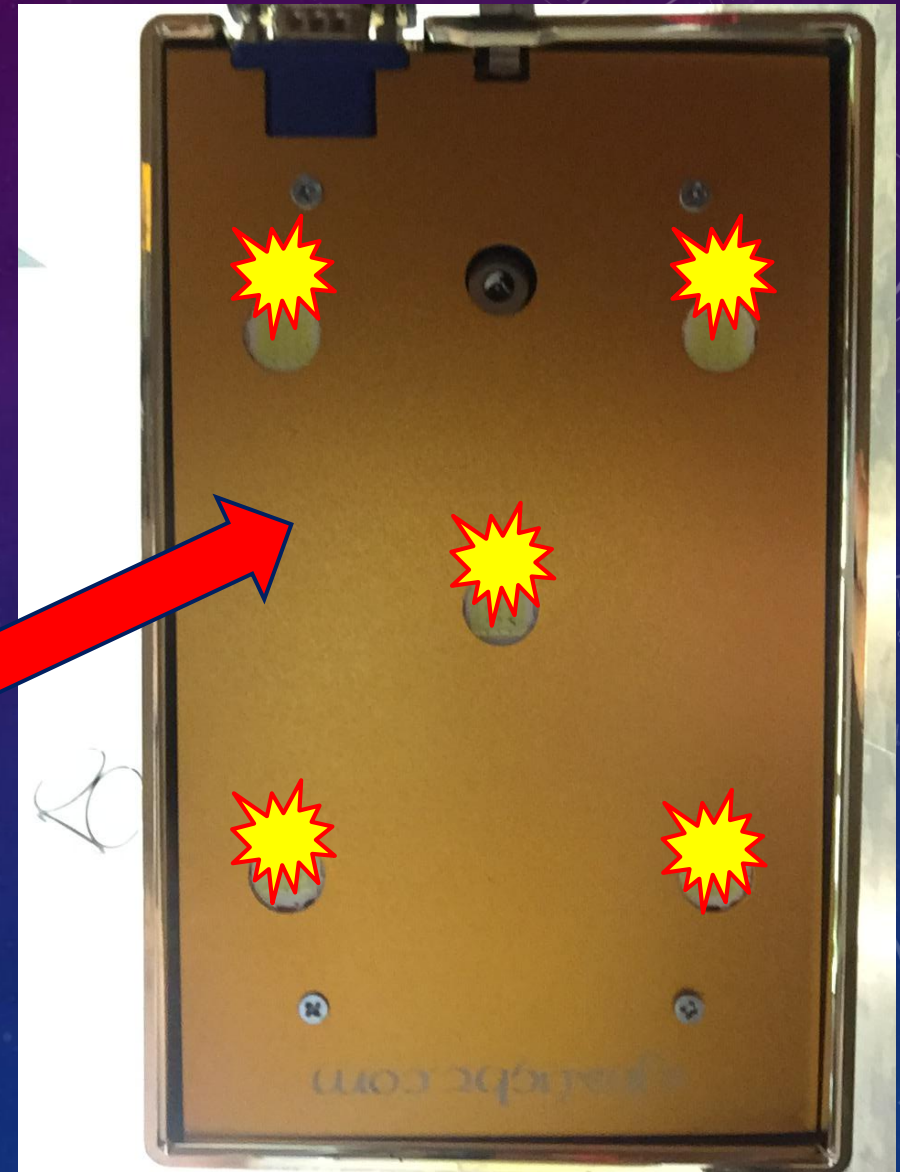




## NEURO LIGHT TECHNICAL SET UP

- There is also a “**hole**” in which to **insert** the “**plug**” for the **power source**.

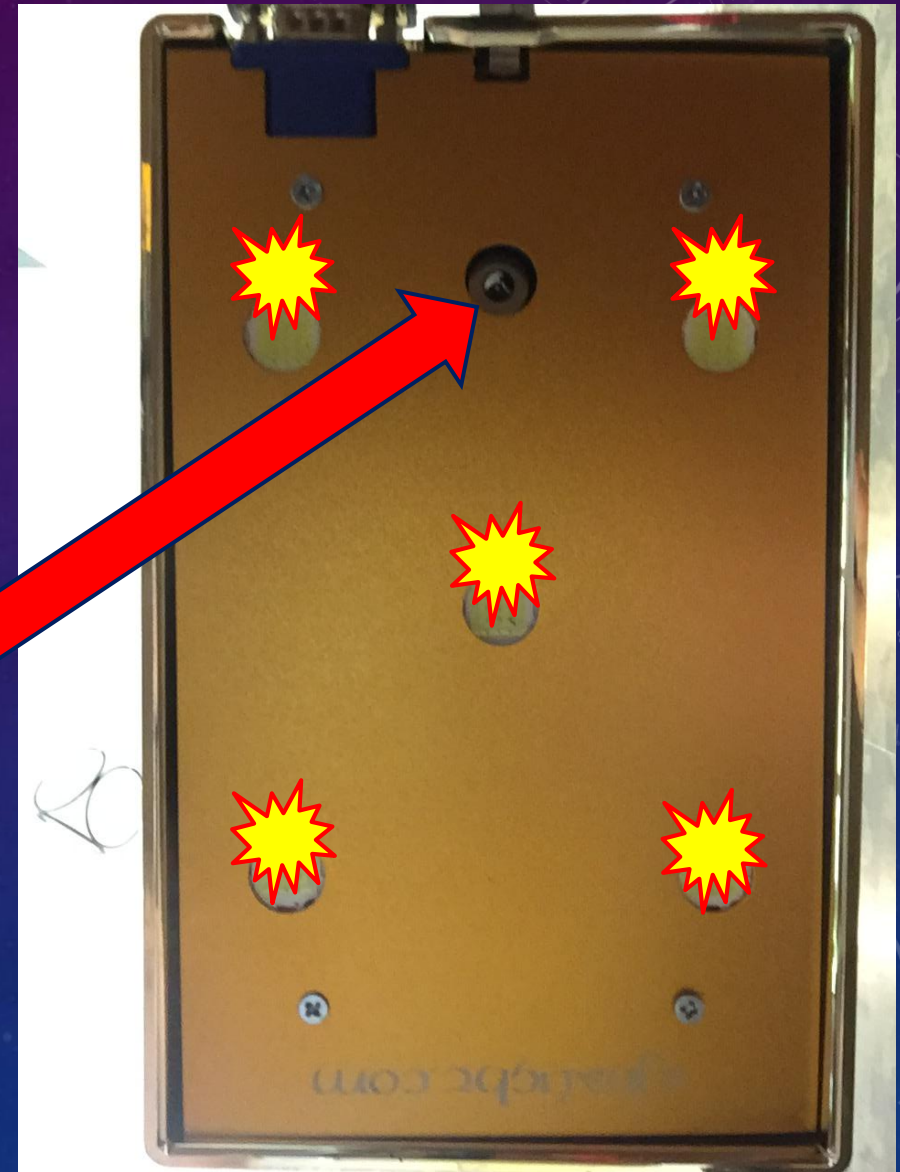

## NEURO LIGHT TECHNICAL SET UP


- Here you see the **power supply** for the Neuro Light **Golden Pyramid**.
- Notice the “**plug**” that goes into the **underside** of the **Golden Pyramid** in the “**hole**”.

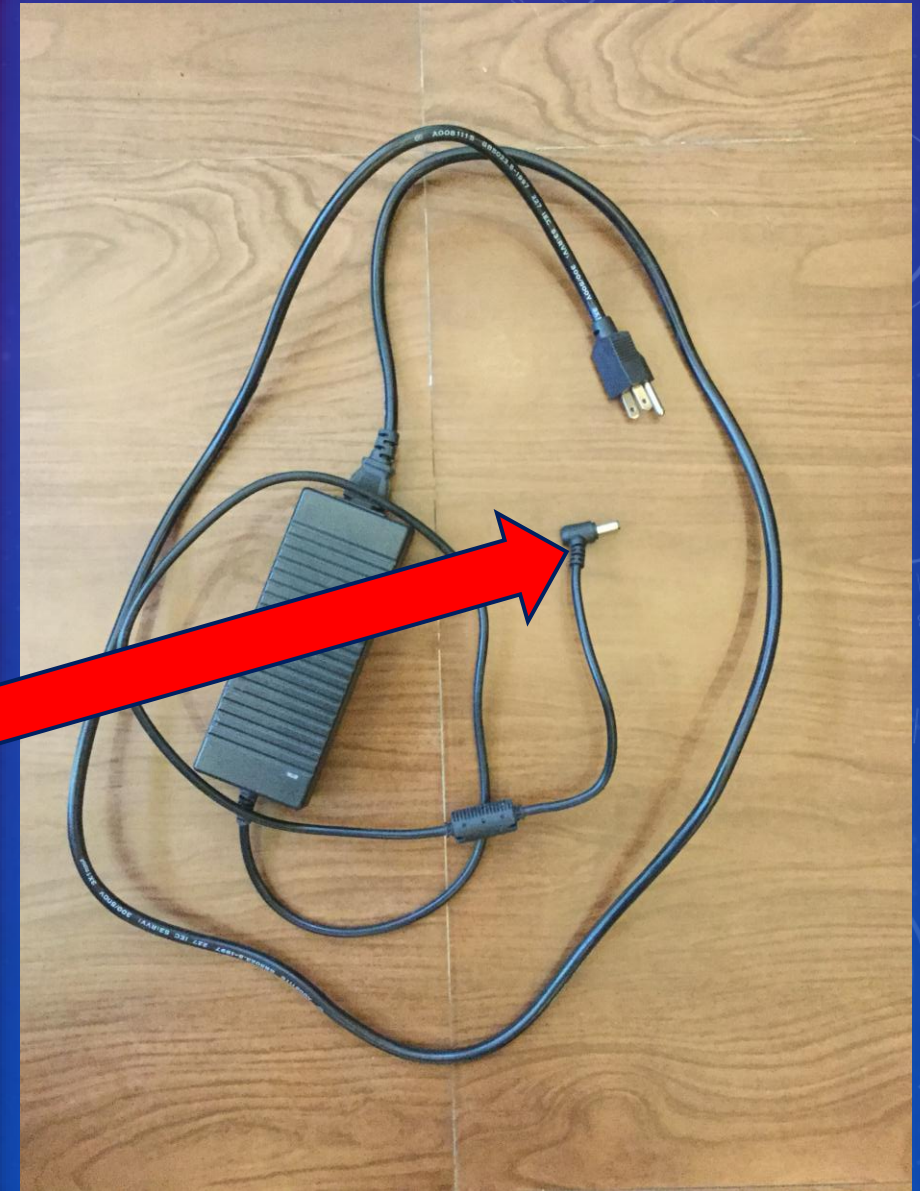



## NEURO LIGHT TECHNICAL SET UP

- Here you see the “**plug**” of the **power supply** inserted into the “**hole**” to give the **Golden Pyramid electrical power**.

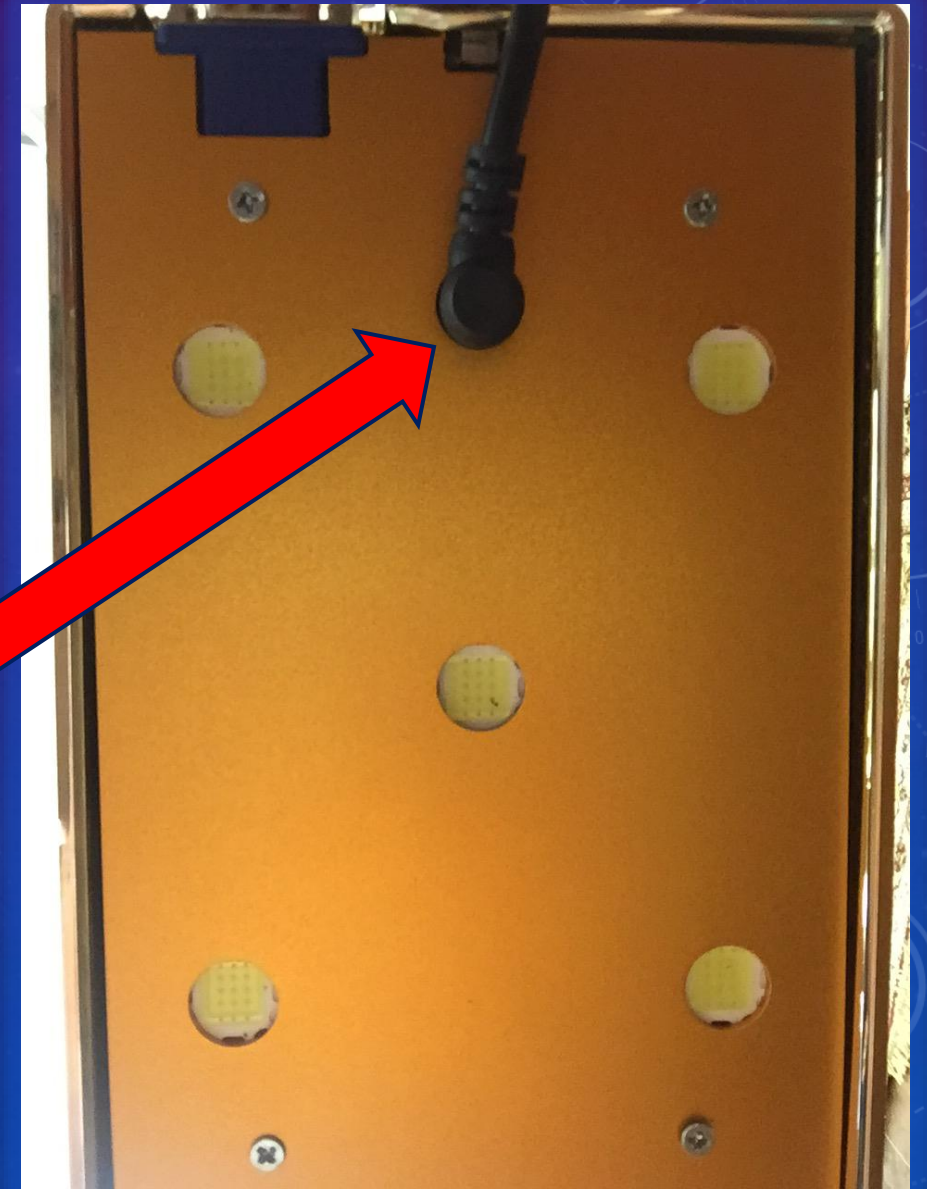

## NEURO LIGHT TECHNICAL SET UP

- You know that your **Golden Box** is **“On”** and has **power** when you **see the bright blue LED** shining at the side of the **Golden Box.**





# NEURO LIGHT

## TECHNICAL SET UP

- You should **protect your Neuro Light** with a **power cord** designed with a **surge protector** which is easily available.
- It also makes for **easy “On/Off”** with just a switch that **tells you** when **power is “On”**.



## NEURO LIGHT TECHNICAL SET UP

- Notice that the **Golden Pyramid** is **connected** to the **tripod** with a **screw type bolt**.
- You must **supply your own tripod** – a tripod is easy to purchase and is **not supplied in your purchase**.





## NEURO LIGHT TECHNICAL SET UP

- Here you see a **typical tripod** set up for the **Neuro Light**.
- Notice the **cord linking** the **Golden Pyramid** with the **Golden Box**.



# NEURO LIGHT TECHNICAL SET UP

- The **Golden Box** is **linked** to the **Golden Pyramid** with this **safe-screw connector**.
- The cable used is a **custom VGA cable**.






## NEURO LIGHT TECHNICAL SET UP

- Another view of the **Golden Box** **linked** to the **Golden Pyramid** with this **safe-screw custom VGA cable**.



## NEURO LIGHT TECHNICAL SET UP


- Here you see the **Golden Box** as well as a pair of **headphones**.
- Notice the **dark blue custom VGA cable** that **links** the **Golden Box** to the **Golden Pyramid** (shown before).

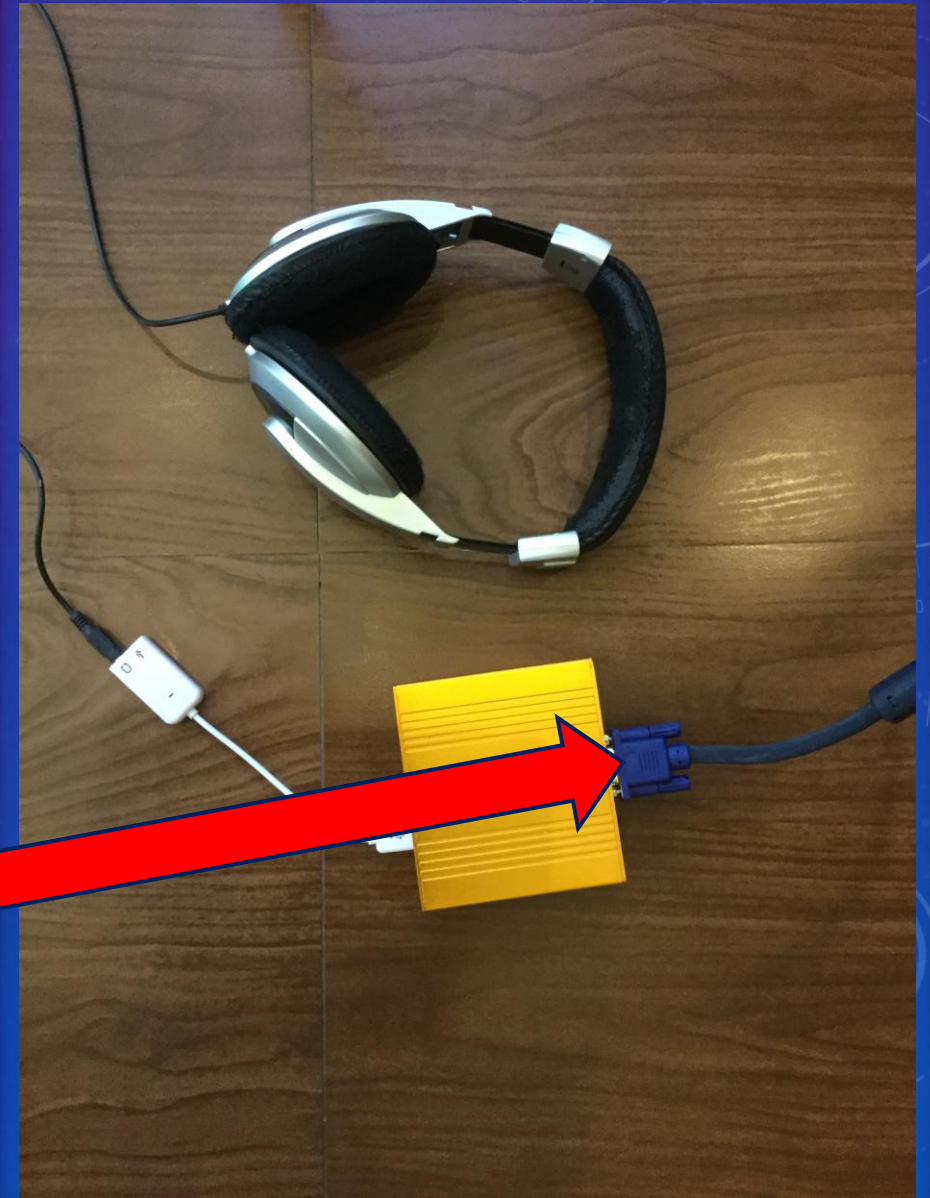




## NEURO LIGHT TECHNICAL SET UP

- Here you see the **Golden Box** and the **dark blue custom VGA cable** that **links** the **Golden Box** to the **Golden Pyramid**.

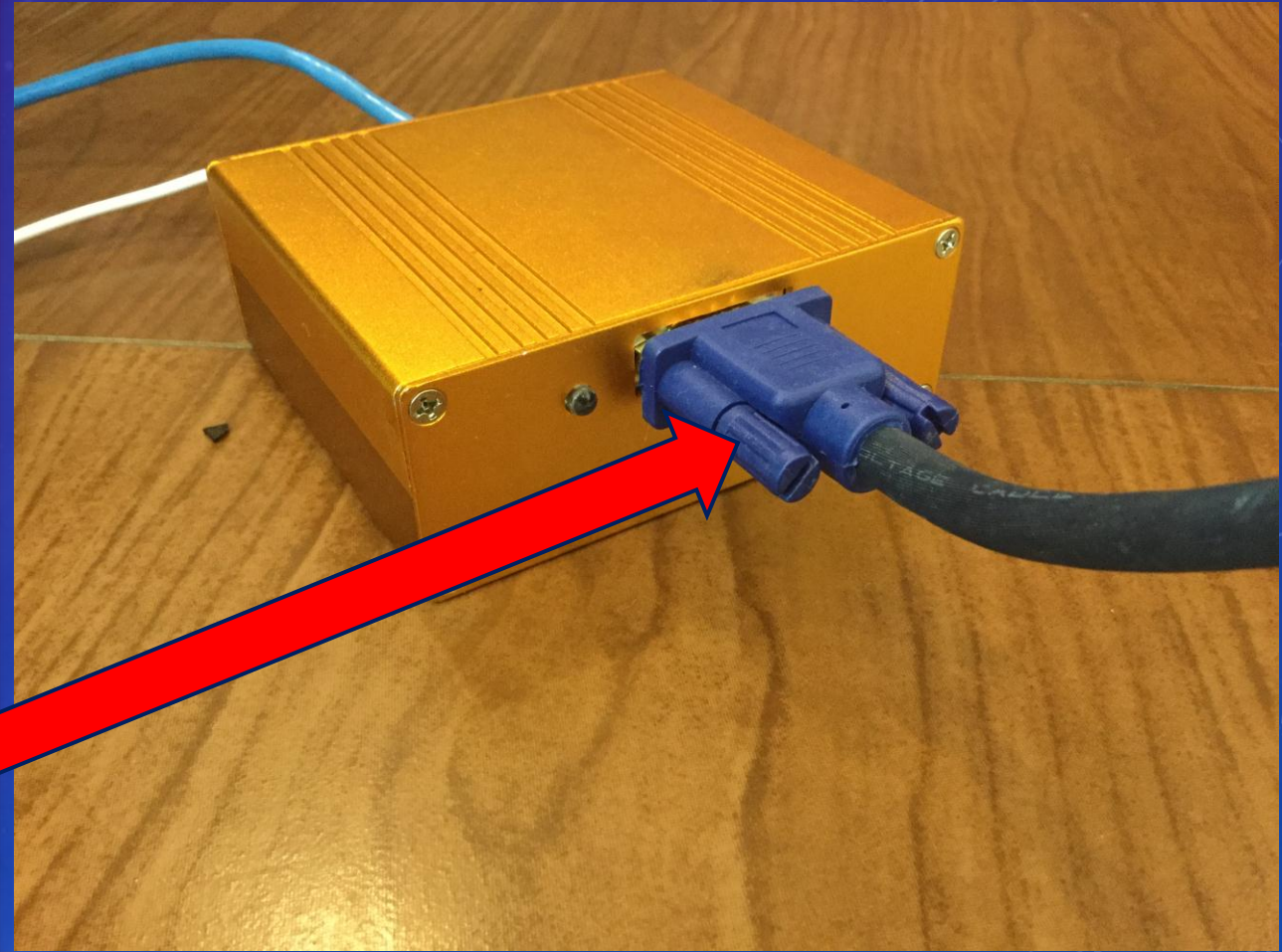

# NEURO LIGHT TECHNICAL SET UP


- You can also see the **headphones link** with the **Golden Box**.

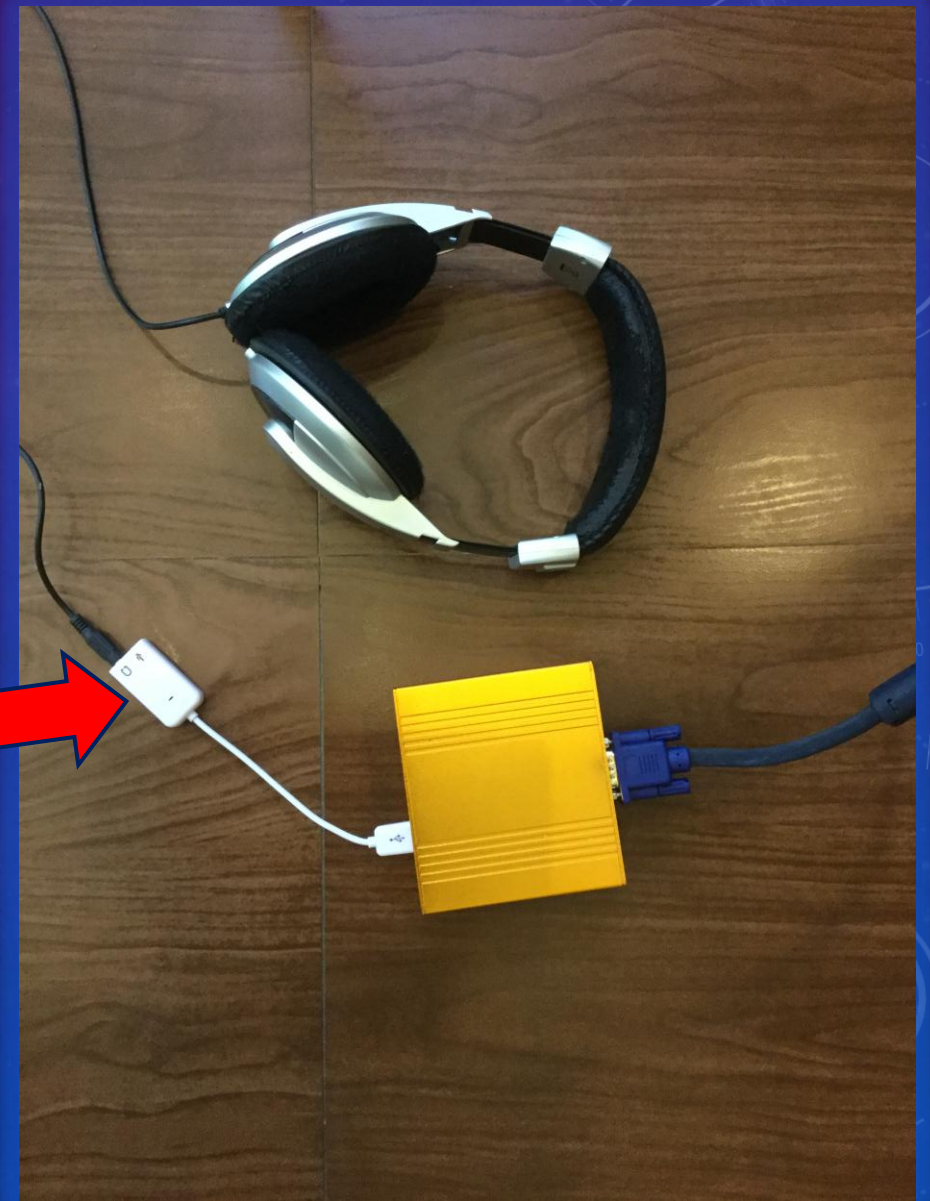




## NEURO LIGHT TECHNICAL SET UP

- **Plug** your **headphones jack** into the **white connector** at the **hole** with the **headphones symbol**.

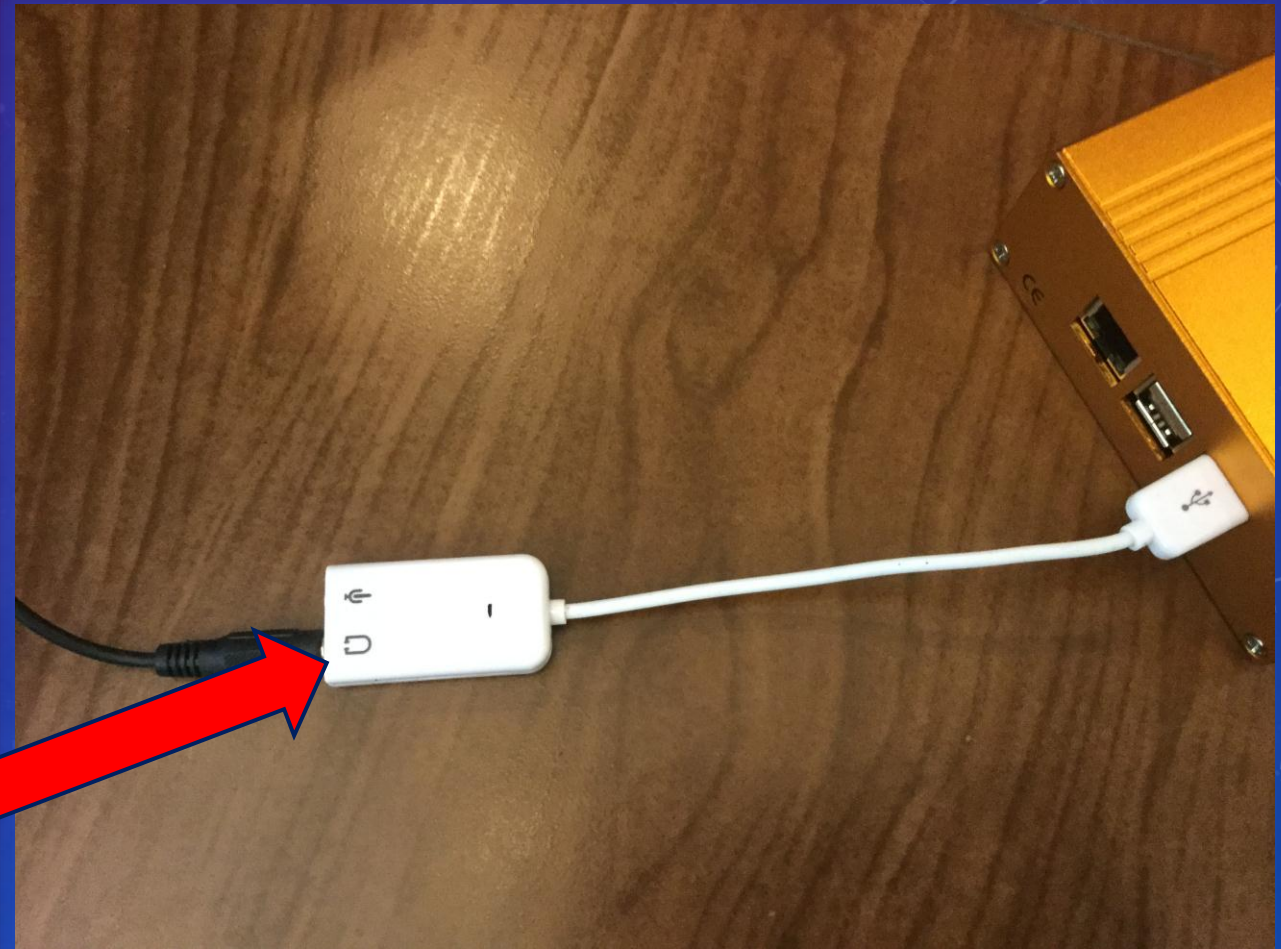

## NEURO LIGHT TECHNICAL SET UP

- The **Golden Box** is actually a **micro computer** which contains an SDcard drive with all the **Neuro Light software**.
- It also has the “**local WiFi**” that **talks directly** with your controller **tablet, smartphone or computer**.

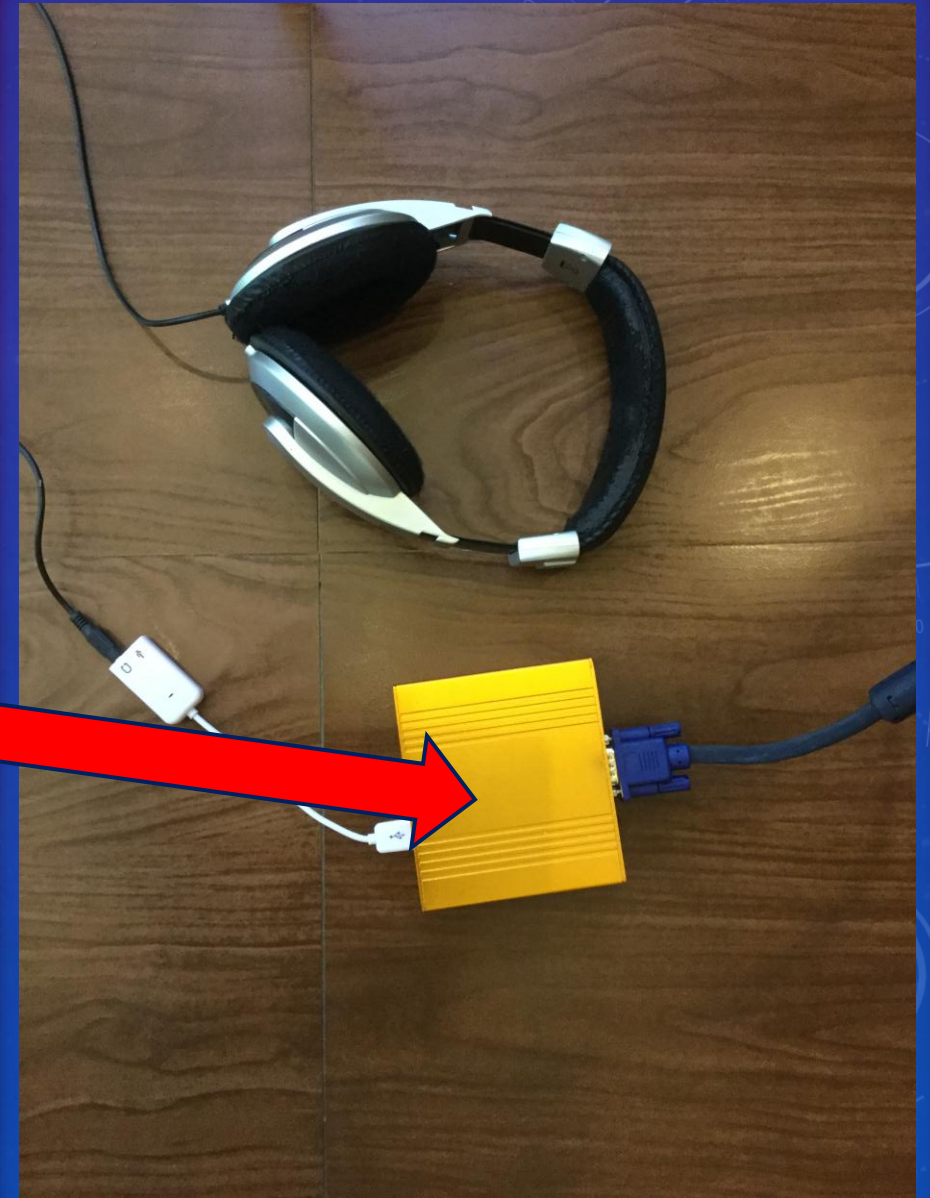



## NEURO LIGHT TECHNICAL SET UP


- Here you see the **headphones connector** linking your **headphones** to the **Golden Box**.
- You must **supply your headphones**.
- Choose a **model** that does **not have too much bass** in its design.





## NEURO LIGHT TECHNICAL SET UP

- This is also where you can **connect** an **Ethernet cable** if you want.

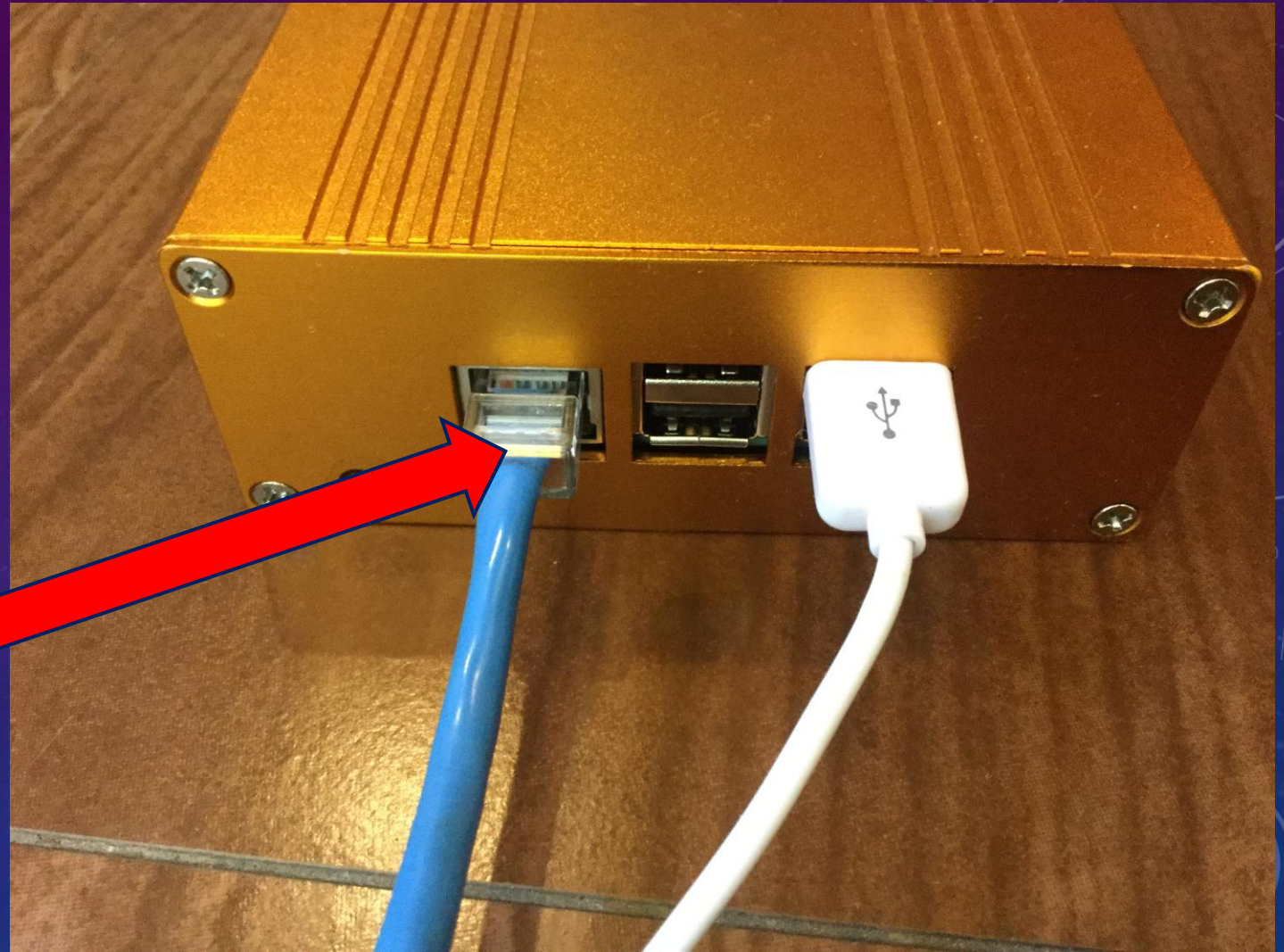



## NEURO LIGHT TECHNICAL SET UP


- If you **choose** an **Ethernet connection** instead of WiFi, you must **connect** the **Ethernet cable** into your **internet router** and **not your computer**. *(more about that later).*



# NEURO LIGHT TECHNICAL SET UP

- When you want to **connect** your **Neuro Light** to your **tablet, smartphone or computer**, you can choose the **WiFi style** or the **Ethernet**.


- Here is the **WiFi style**.

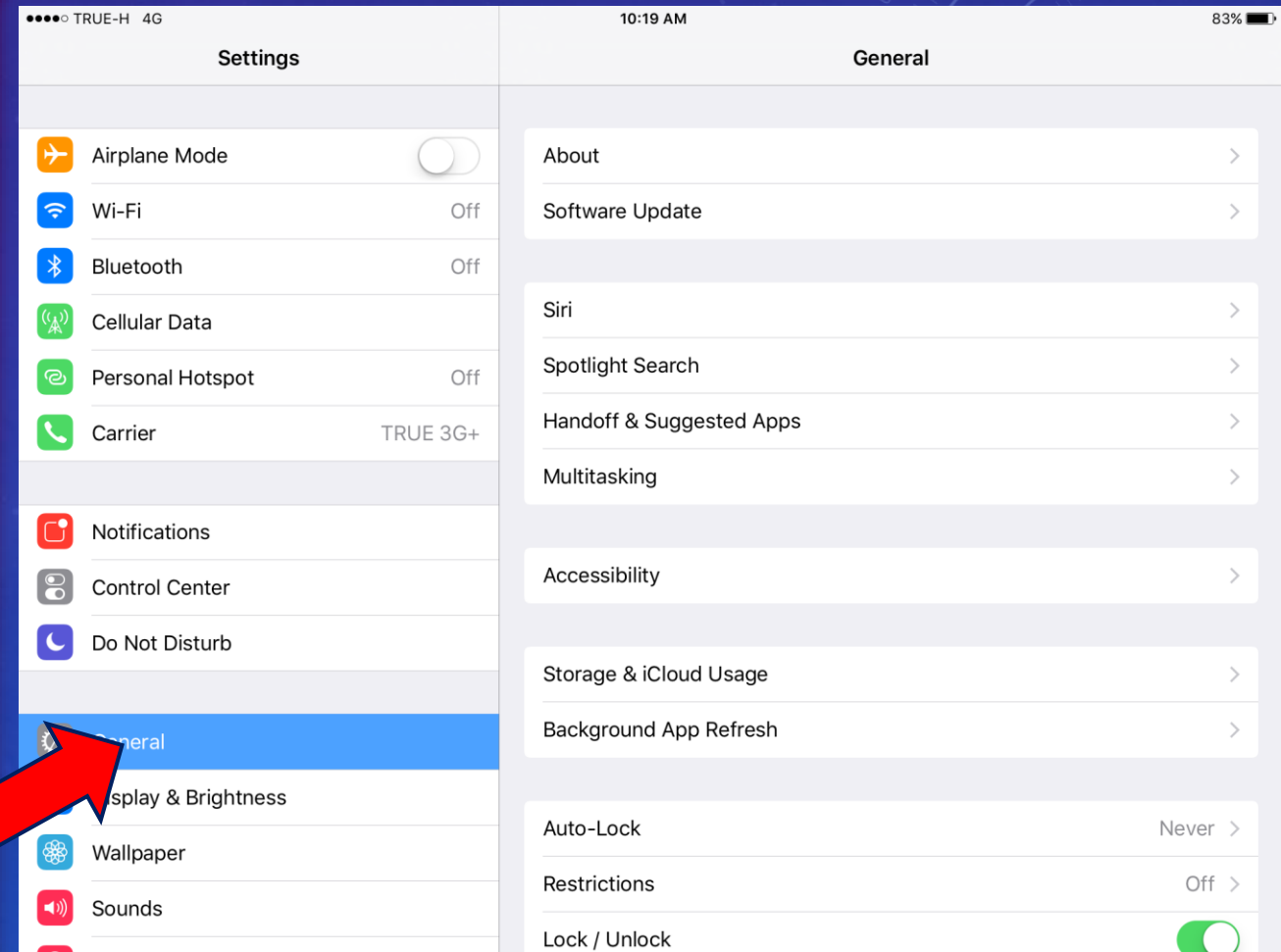




# NEURO LIGHT TECHNICAL SET UP

- **TURN ON** the **power** to your **Neuro Light** and **wait a moment.**
- Then **open your device** in **Settings.**

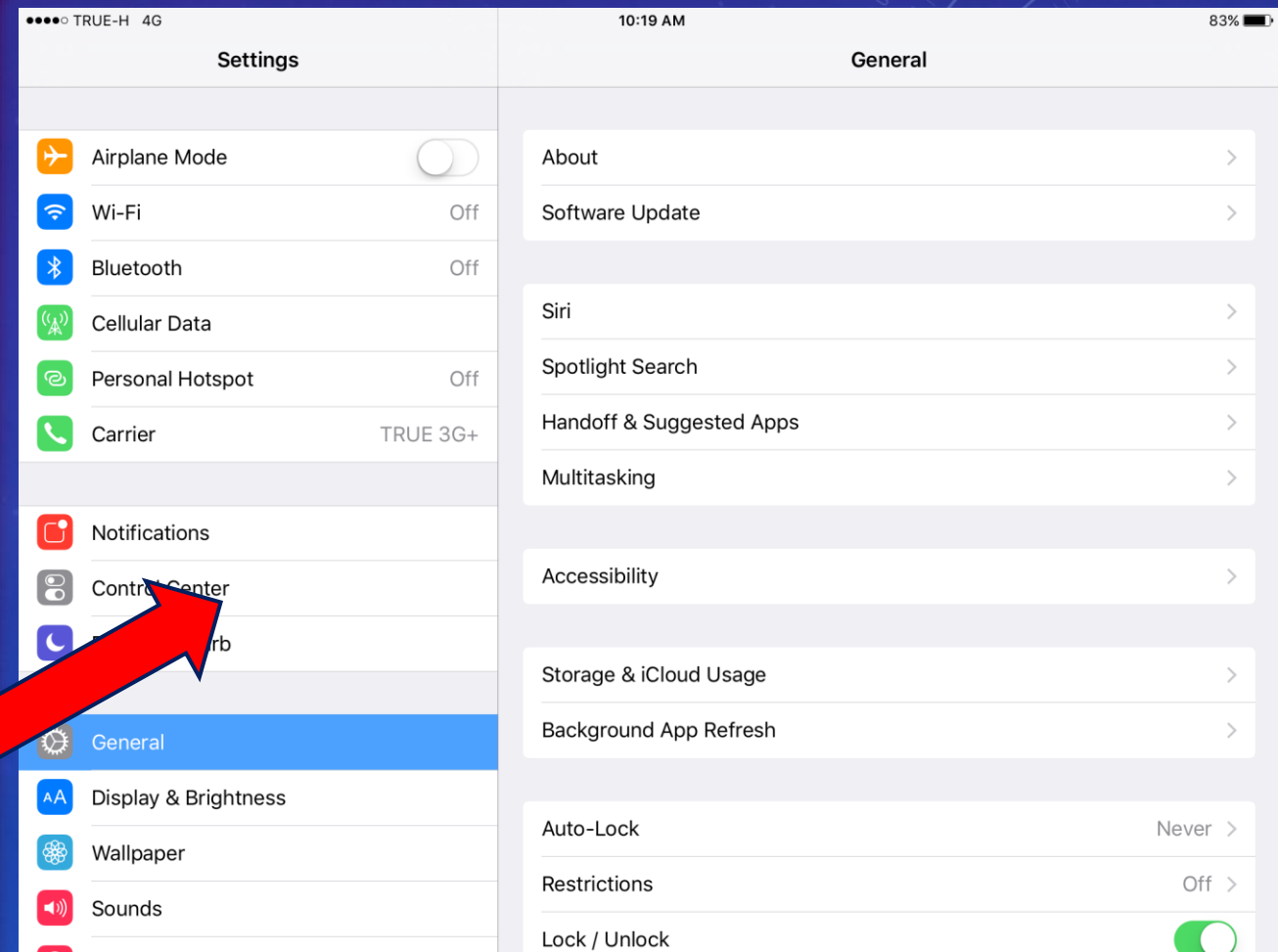

# NEURO LIGHT TECHNICAL SET UP


- Go to your **WiFi settings**.
- Understand that you **DO NOT need to connect to any public WiFi**.
- This is a **LOCAL WiFi connection** just with your **Golden Box** and it is **free** and does **NOT connect** to the public Internet.

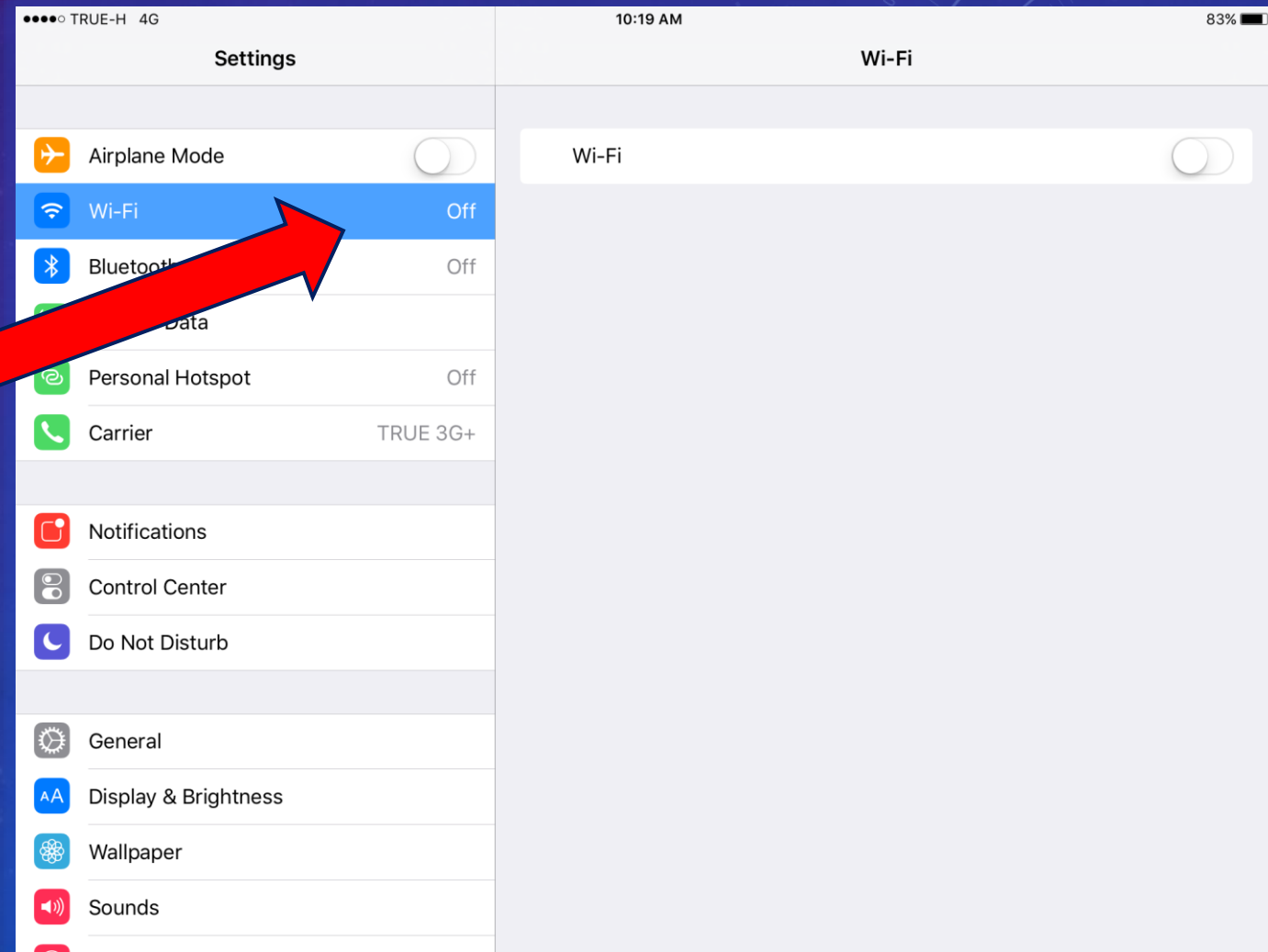




# NEURO LIGHT TECHNICAL SET UP

- **Turn your WiFi “ON”.**
- **Select the “NeuroLamp”** WiFi connection which is coming from your **Golden Box.**

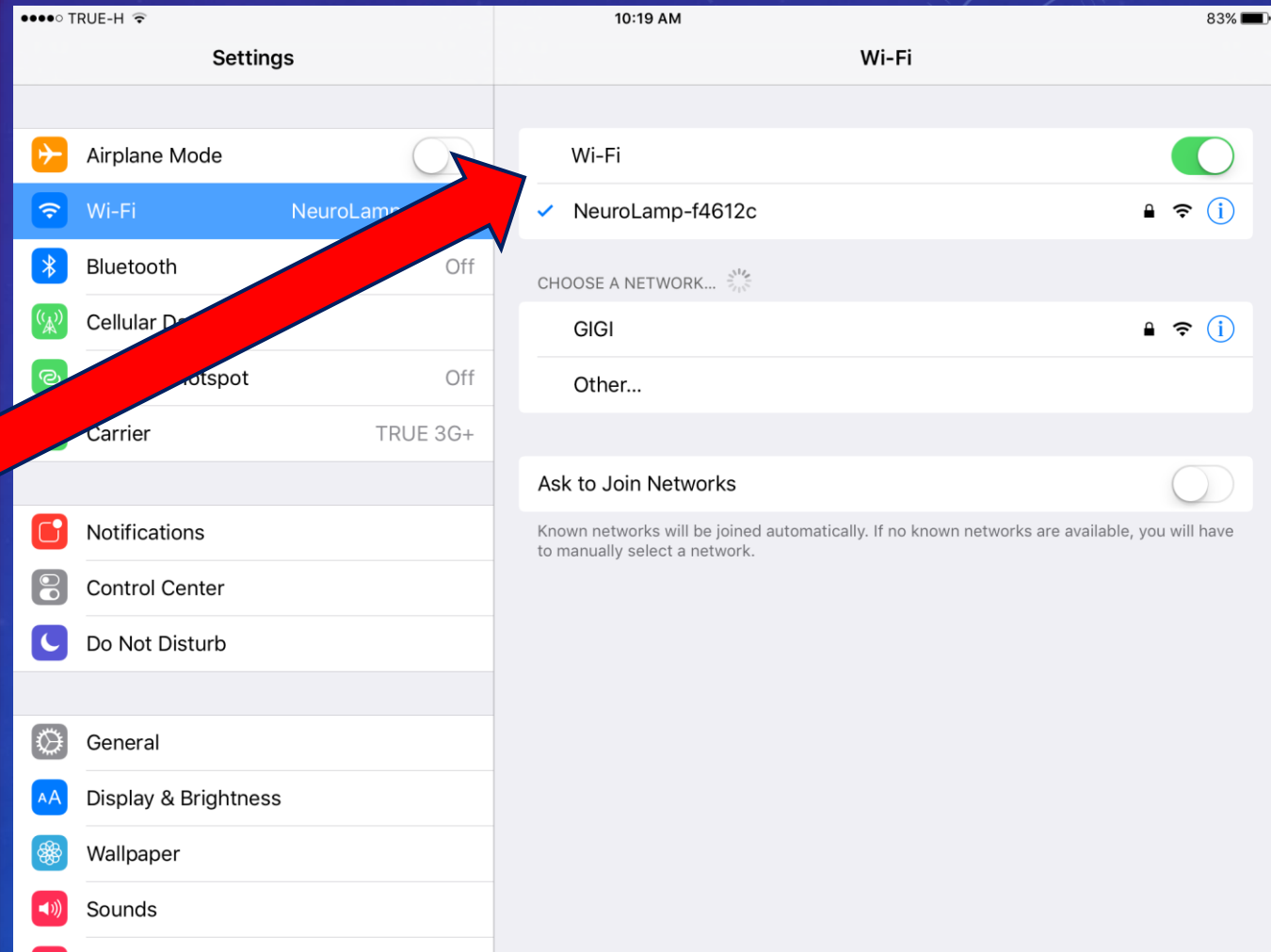

# NEURO LIGHT TECHNICAL SET UP


- If it is **your FIRST time to connect** with this tablet etc., you will be **prompted to enter the Neuro Light WiFi password** which is **pineal777**.
- **Once you enter the password**, you will be **connected** thru this **Local WiFi** to your **Neuro Light**.

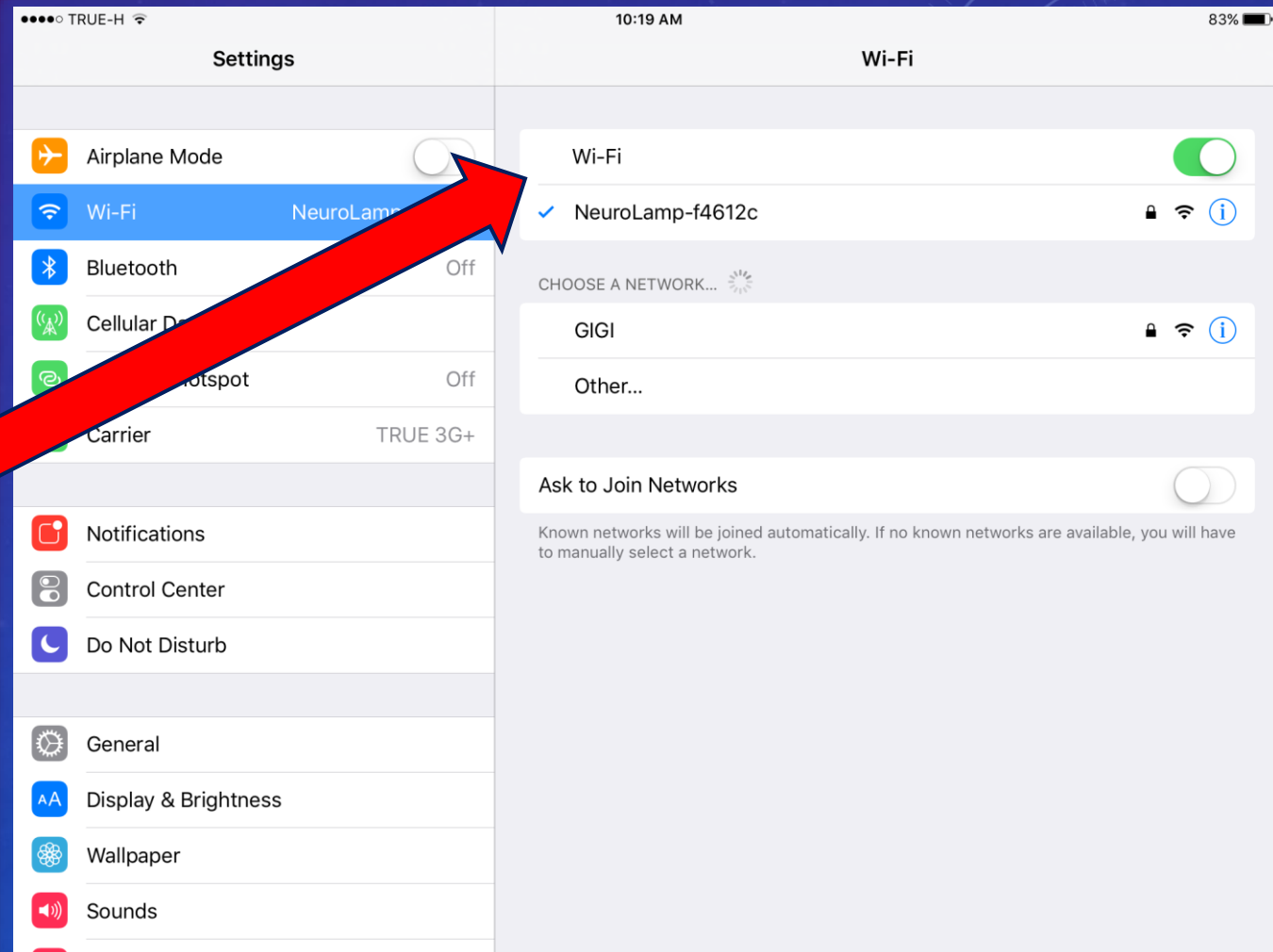




# NEURO LIGHT TECHNICAL SET UP

- Once you have **connected to Local WiFi**, now **open a browser** such as **Google Chrome** or Safari.
- **Enter the URL nl.com**

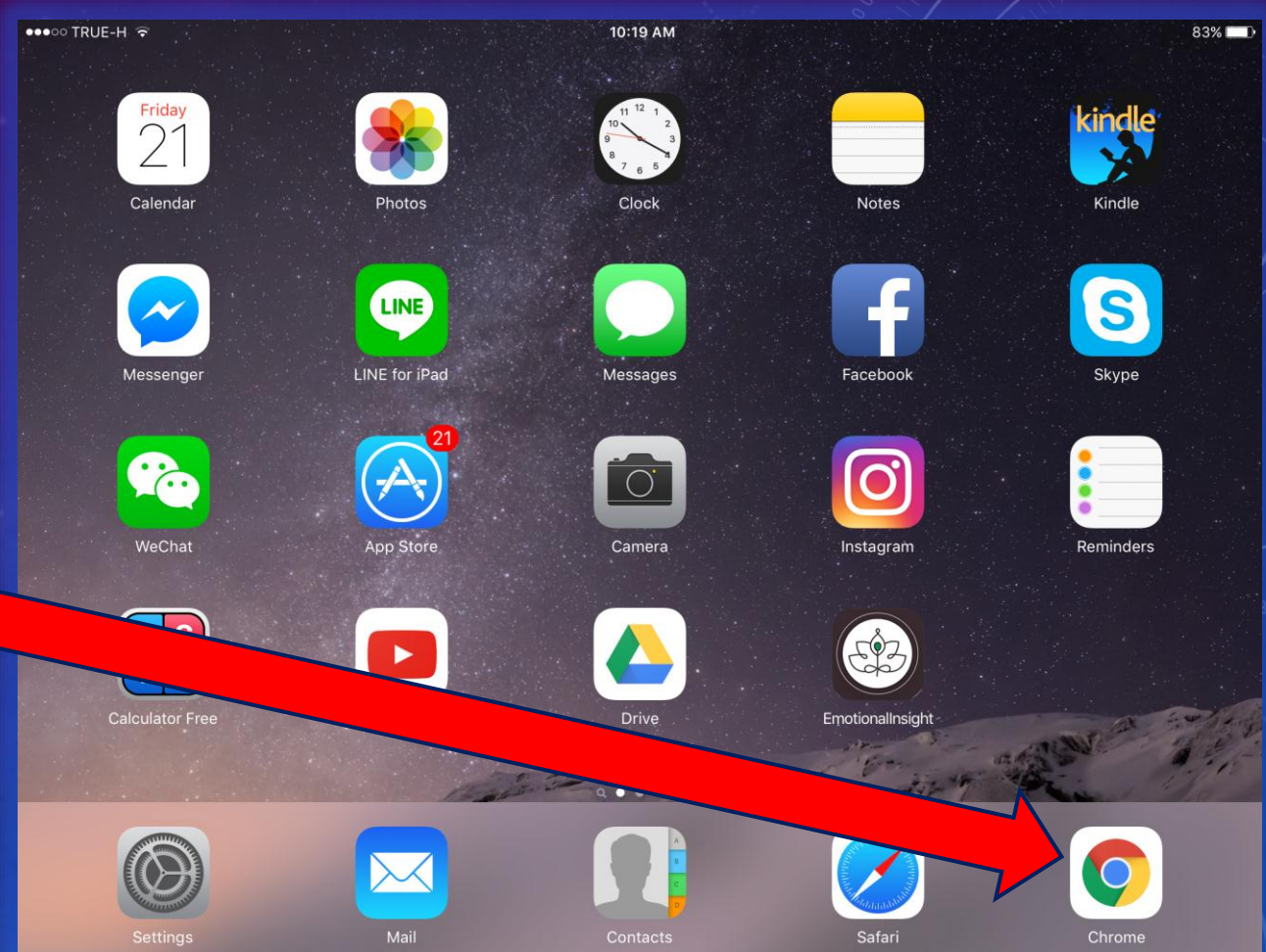

# NEURO LIGHT TECHNICAL SET UP


- If you followed these **simple steps**, the **Neuro Light control panel and User Interface** will show like this.
- **You are ready to use the Neuro Light.**

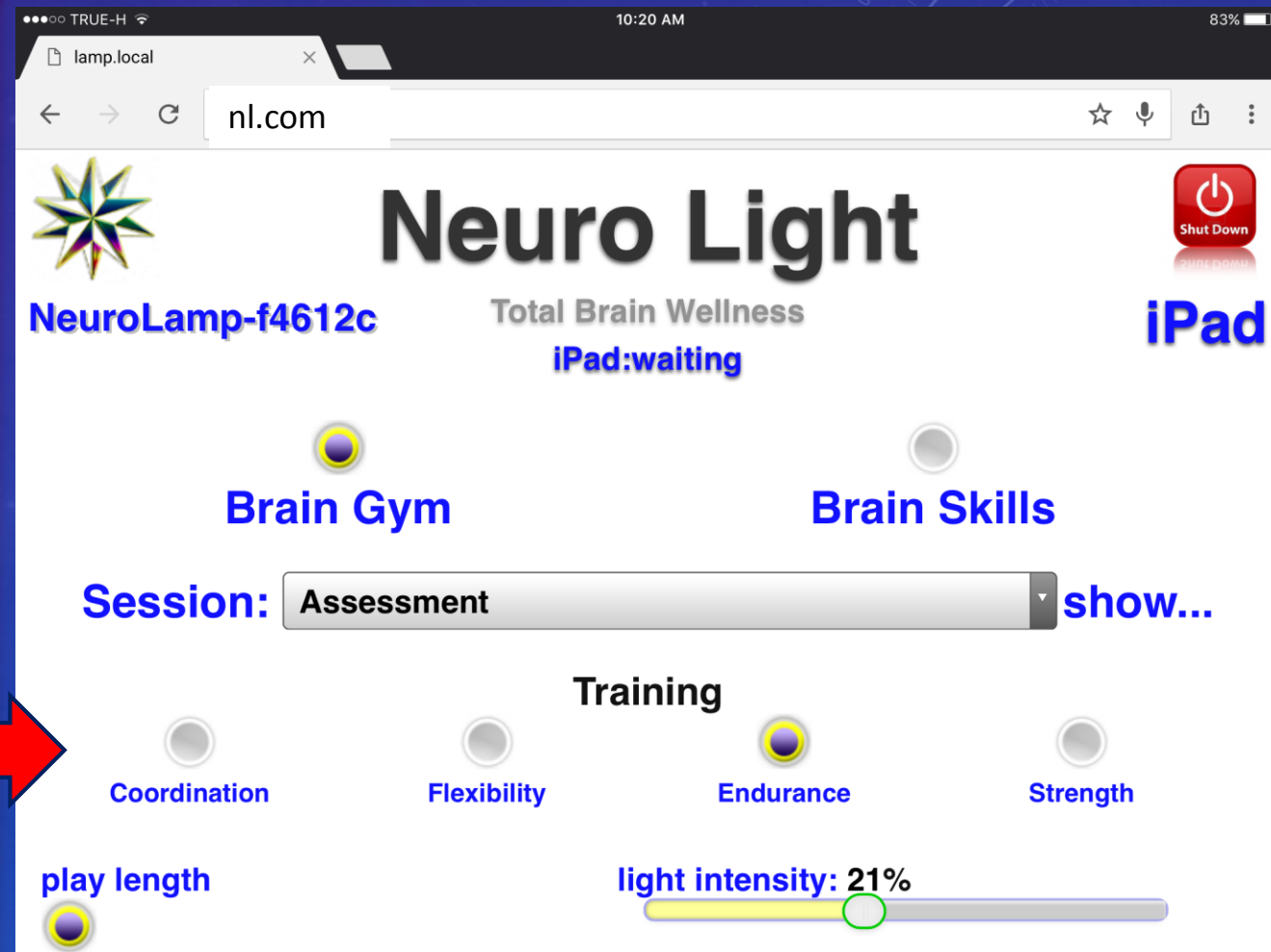



# NEURO LIGHT TECHNICAL SET UP

- If you **prefer** an **Ethernet connection** instead of a WiFi, **plug in your Ethernet cable** as shown earlier in this presentation.
- **Open** up your **browser** and **enter 192.168.1.111** and **press “enter”** to **directly connect** to your **Neuro Light control panel**.

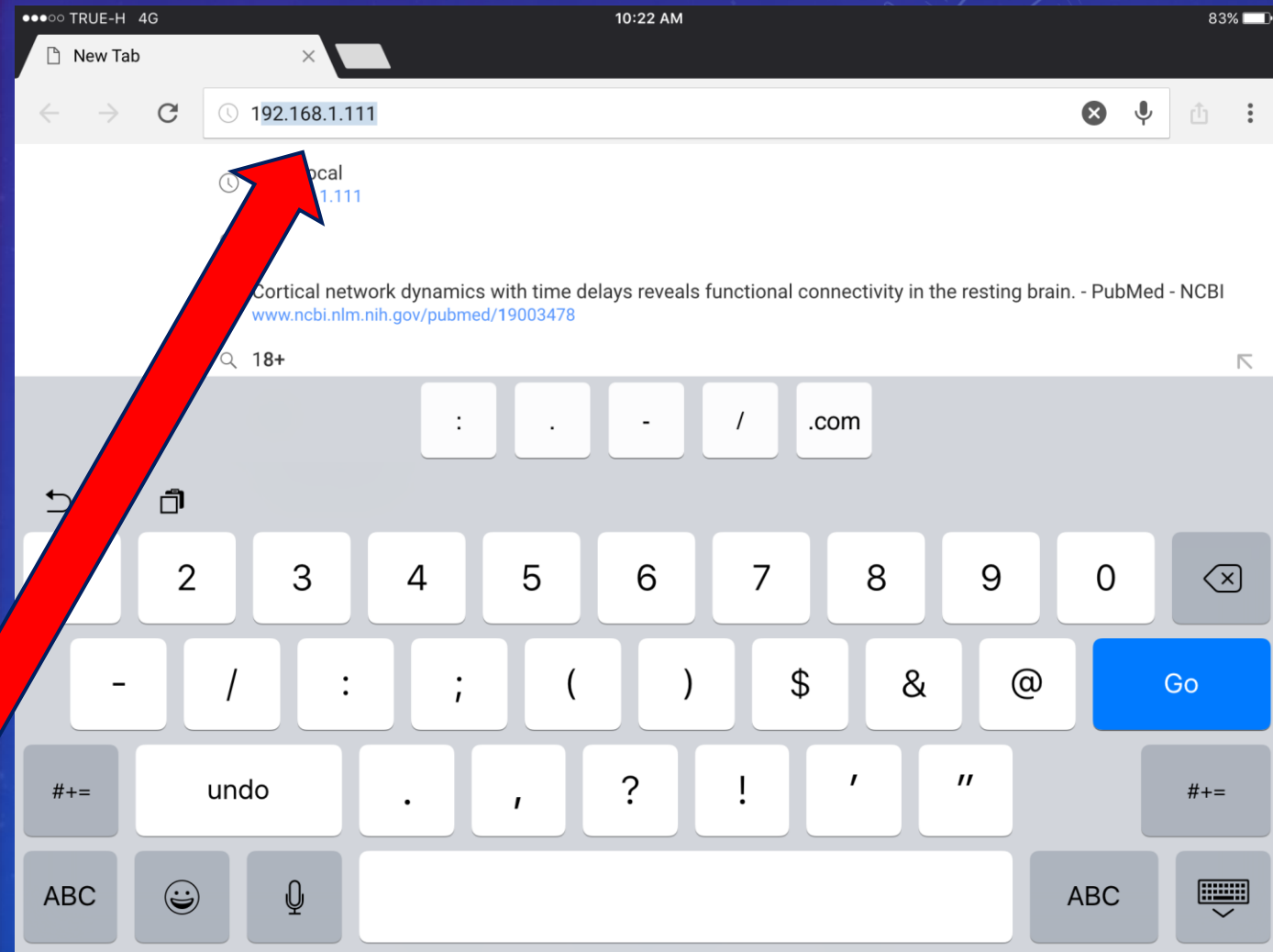

## NEURO LIGHT TECHNICAL SET UP

- **Position** the **Neuro Light** about **30 cm** away from the head.
- You may use a **comfortable lounge chair** or **treatment/massage table**.
- **Enjoy!**

